



Scuba Spec, Inc.
220 Richland Ave., W.
Aiken, South Carolina 29801
Voice: 803-292-6485
Fax: 803-642-5622
Email: scubaspec@gmail.com

The 137 EBP Scuba Spec, or Snorkel Spec, is a prescription spectacle insert specifically designed for the visually deficient diver. The 137 EBP Scuba Spec attaches to the mask faceplate via a suction cup and can be easily inserted or removed from the mask at the diver's discretion. The same prescription powers used in the diver's habitually worn glasses can be used in the Scuba Spec insert (single or bifocal vision).

Care and Use

- When cleaning the glass surface of the mask, use clean water or a mild soapy solution. Do not use window cleaner or an alcohol-based solution.
- To prevent fogging of the faceplate and Scuba Spec lenses, a good anti-fog solution should be applied before diving or snorkeling. Anti-fog solution can be obtained from your eye-care professional or dive shop.
- Rinse the mask and Scuba Spec with clean water after each use.
- The screw component of the suction cup assembly is coated with a clear nail polish to protect it against rusting. Thoroughly dry the suction cup assembly after each dive to prevent rusting.
- The nylon Scuba Spec frame should be hydrated every six months by soaking in tap water for 24 hours. This will keep the frame from drying out and cracking.



Note to Optician

- The Scuba Spec frame should be hydrated every six months by soaking in tap water for 24 hours. This will hydrate the frame and ensure its malleability. Be sure to dry the suction cup assembly after hydration to prevent rusting of the metal screw component.

Scuba Spec Warranty (excluding lenses)

Four years against breakage if properly cared for. Proper care entails (1) soaking the frame in tap water for 24 hours every six months to hydrate the nylon material and (2) drying the suction cup assembly after each submersion to prevent rusting of the metal screw component.